

How to apply and remove the TR Seal.



Medical Crisis Response System



1 Hold seal strip as shown. On seal being detached, place thumb only on seal-tab, NOT on tear-off rod. Now bend seal-tab and pull it apart from strip.



2 Insert bail through the grommets in the CLOSED double flap in the TR Kit and insert the end into the locking barrel of the seal.



3 Pull on bail until it engages the locking mechanism. It is NOT necessary to cinch it up so tight that the material buckles. The TR Kit is now tamper resistant.



4 Remove by pulling the tear-off rod away from the seal-tab and out of the grommets. The TR Kit cannot be re-sealed using the original seal.